

How to use TUBE EVAC™

1. Place the tubing in the groove of the tube-evac close to where it exits the skin.
2. With your other hand, hold the tubing firmly between the skin and the tube-evac.



3. Firmly squeeze the tube-evac and pull it towards the collection bulb.
4. Advance other hand as necessary to hold tubing.

5. The tube-evac rolls down the tubing very easily with a small but firm squeeze to evacuate small fibrinous material and or clots to maintain a functioning drain.



TUBE-EVAC

- Tube-Evac:** A device created by the husband of a breast cancer survivor. The device is designed to "clear" tubing attached to a drain placed in a surgical site.
- Tube-Evac:** Manufactured in USA under strict quality controls.
- Tube-Evac:** Holds: Design Patent
Utility Patent Pending
International Provisional Patent
- Tube-Evac:** Designed to clear 7-10 French and 15-19 French gauge tubing.
- Tube-Evac:** Prevents over-stretching and distortion of tubing.
- Tube-Evac:** Offers a quick, easy and safe way for doctors and nurses to clear tubing.
- Tube-Evac:** Single patient use only. To be discarded once drains are discontinued.
- Tube-Evac:** Creates peace of mind for patient and caregiver with safe, effective ease of use.
- Tube-Evac:** Web site with demonstration video, manufacturer and order information available at tube-evac.com or tube-evac@tampabay.rr.com.

Vance Shaffer - Inventor/President
Crystal River Medical Devices
P.O. Box 1538
Crystal River, Florida 34423
352-257-8586

